

SaniSchutz Flächendesinfektionsmittel

1 PRODUCT DESCRIPTION

Surface disinfectant cleaner for all water-proof surfaces, for use in commercial and household areas, especially in medical areas. Ready-to-use solution based on quaternary ammonium compounds. Acts effectively as anti-bacterial, fungicidal and virucidal. Suitable for conductive surfaces.

The combination of the active substances with cationic and amphotheric surfactants enables a high surface activity and has a good cleaning effect with a comparatively low pH value. Tested in compliance with the requirements of the testing methodology of the VAH.

2 AREAS OF APPLICATION

For use in commercial and household areas, especially in the area of public health services, as well as in food and animal fodder areas (BAuA-Registration-Nr. N-90882). Suitable for the wiped disinfection of non-porous, hard, washable (water-proof) surfaces and objects (walls, tables, chairs, wash-basins, etc.) in hospitals, medical practices, public institutions (schools, sports facilities, swimming pools) and in the area of food production and processing. Applications in general clinical practices and for use in the food processing industry.

3 APPLICATION

Use biocidal products with caution. Before use always read the label and product information !

For all areas of application the ready-to-use solution is sprayed onto the surfaces and wiped wet. Make sure that the entire area is moistened. The prescribed contact time of 15 minutes must be observed (see paragraph "Special Notes").

Please note: do not mix Dr. Schutz SaniSchutz surface disinfectant cleaner with other cleaners and/or disinfectants, as its described effectiveness is no longer given.

4 CONSUMPTION

approx. 500 ml on 10 sq.m.

5 STORAGE

SaniSchutz Flächendesinfektionsmittel

Store the disinfectant cleaner concentrate in its tightly sealed original container in a cool, dry place. Protect against heat and direct sunlight. Store locked up and out of the reach of children. Do not store near foodstuffs, drinks or animal fodder. Will keep for 36 months in the original container from the date of manufacture.

6 TECHNICAL DATA

Contains substances: less than 5% cationic surfactants, less than 5% amphoteric surfactants, Disinfectants quaternary ammonium compounds, benzyl-C12-18-alkyldimethyl.

pH-value: approx. 9 (ready-to-use solution)

ADR/RID: not a hazardous good as defined by transport regulations.

CLP: not a hazardous substance as defined by hazardous substances regulations.

Even when using low pollution products the customary safety measures are to be observed. Completely emptied containers can be disposed of in household waste, or taken to a recycling centre.

7 SPECIAL NOTES

The formula is registered in the disinfectants list of the VAH and in compliance with its requirements has been proven effective under the following conditions:

Surface disinfection of non-porous surfaces for prophylactic purposes in hospitals and medical practices as wiped disinfection, mechanically under increased organic pollution (anti-bacterial and levurocidal):

Recommended contact-time: 15 mins.

8 CROSS REFERENCE

Our following publications in their updated form are valid as an integral part of this fact sheet:

- General notes on the application of Dr. Schutz surface products
- Safety data sheet available on request.

The quoted printed matter, as well as this product information in its updated form, is available on our homepage www.dr-schutz.com or on request from the address quoted below.

SaniSchutz Flächendesinfektionsmittel

Dr. Schutz - We Care About Floors

Dr. Schutz GmbH
Holbeinstraße 17
53175 Bonn
Tel.: + 49 (0)228 / 95 35 2-0
Fax: + 49 (0)228 / 95 35 2-29

E-Mail: zentrale@dr-schutz.com

Schweiz

Reinwall GmbH
CH 9500 Wil
Werksvertretung Schweiz und Liechtenstein:
Floor Concept GmbH
Bachstrasse 29
8912 Obfelden
Tel. +41 (0)44 533 45 00
Fax +41 (0)44 533 45 01
E-Mail: info@floorconcept.ch
www.floorconcept.ch

Österreich

Michael Neubauer
Tel. : +43 (0)664 3562985
Fax: +43 (0)463 265526 8178
E-Mail: mne@dr-schutz.com